



COLLINSON

Back Home PCR Travel Test



IMPORTANT: Please read these instructions carefully before you take your test.

Welcome to your Back Home PCR travel test

This user guide takes you through the testing process and what you'll need to do to get your results.

There is a separate instruction leaflet in the kit to tell you how to register and take your travel test.

If you have any questions, get in touch with the Collinson support team at:

consumersupport.collinsonassistance.com

STEP 1


When you're back home



The UK Government requires all travellers aged five and over arriving in the UK to take a PCR test on or before day 2. Your arrival day is classed as day 0.

STEP 2

Before you take the test

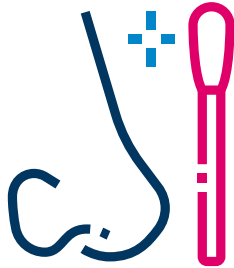

Do not eat, drink or smoke for at least 30 minutes before the test.

The test should take approximately 10 minutes to read the instructions, set up and conduct the test and a further 5 minutes to package up your sample to send to the lab.

STEP 3

Sending your test back



Once you've taken the test, make sure you place your sample with the absorbent pad into the biohazard bag and seal it.

Then place it into the prepaid postal rigid pouch, making a note of the registered tracking information before sending.

Send your sample immediately to the lab by Royal Mail from:

- Parcel postbox
- Priority postbox
- Post Office counter
- Post Office Customer Service Point (CSP).

To find your nearest one at:

[royalmail.com/services-near-you](https://www.royalmail.com/services-near-you)

STEP 4

Getting your results



A notification will be sent to you once we've received your sample.

We will then process your sample and send your results to your email address.

Collinson will then notify the relevant national health authority with your results.

A negative result means that it is likely that you're not infected with COVID-19 and unlikely to infect anyone.

A positive result means that you're infected with COVID-19 and are at risk of infecting others too. Therefore you must self-isolate and follow the latest government and NHS guidelines on self-isolation.

If your result is inconclusive or spoiled or if you have any questions, get in touch with the Collinson support team at:

consumersupport.collinsonassistance.com

Contact us



collinsontest.com



consumersupport.collinsonassistance.com



COLLINSON

All content correct at time of publication October 2021